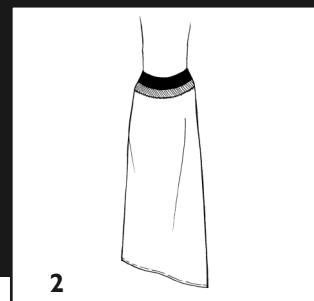
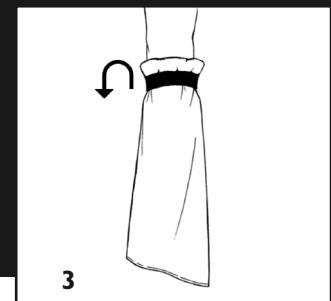
1

2

3

## GB

### Fitting your LimbO

Pull the LimbO over your cast, gripping it by the edge of the seal as shown, and pull it up as high as you can (figure 1).

Once the LimbO is as high as you can get it, gently tug it downwards and the seal will then lie flat against your skin (figure 2).

**On leg models fit a thick sock or similar over the cast foot before fitting the LimbO. This provides cushioning and reduces the risk of damaging the LimbO.**

If the LimbO is to be submerged (especially leg models in the bath) we recommend that any air trapped in the LimbO is removed.

This is done when it is out of the water by carefully lifting the seal and squeezing the air out of the LimbO and then letting the seal go back against the skin.

The LimbO should be worn for no longer than 20 minutes as the trapped air gets quite warm which can cause perspiration and discomfort.

### Removing your LimbO

Carefully roll the seal back on itself as shown so that its inner surface faces outward, and then slide the LimbO off (figure 3).

### Care of your LimbO

Handwash as required with warm soapy water. Avoid folding the seal.

Hang upside down and air dry before storage. Ensure seal is undamaged before re-use.

### Product Safety Instructions

The LimbO should be used according to these instructions for maximum durability:

- Never use the LimbO for anything other than its intended purpose as a cast or dressing protector
- Never use the LimbO as a flotation device
- Do not use the LimbO in recreational water activities. The product can be used for gentle submersion, e.g. in a bath or 'splashing about' in a pool
- Care must be taken not to tread on any sharp objects such as rocks or stones
- Children or persons with special needs using the LimbO should be supervised at all times
- Full leg casts are heavy and severely restrict movement, so for safety reasons you should not go into pools or the sea
- The LimbO is a non sterile product and should not be used on open or undressed wounds or when any danger of infection is present
- If you intended to use your LimbO while under medical care, always obtain advance approval from your medical provider
- The LimbO can be used in hydrotherapy as part of your medically approved treatment

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