rollbuddy®

Indoor Rollator

GMN:
8720726413RB013J

V1.0NL/EN

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EN User manual

Fig. 1:

- A** Brake lever
- B** Removable serving tray
- C** Drawer
- D** Brake cable
- E** Inner frame
- F** Swivel wheel
- G** Comfort foam handle bar
- H** Height indicator
- I** Adjustment knob
- J** Central hinge
- K** Outer frame
- L** Brake wheel
- M** Screw knob
- N** Metal connection plate
- O** Round connection tube

Thank you for purchasing the Rollbuddy indoor rollator. The Rollbuddy is a medical device, this means that the Rollbuddy has been developed and tested according to European standards. Before use, read the manual carefully for optimal and safe use.

General:

Rollbuddy assists standing and walking independently, and facilitates daily activities. The rollator is intended for indoor use only. The drawer and serving tray can be used to store and transport small household utensils.

The use of the Rollbuddy should always be coordinated in consultation with an authorized advisor, doctor or physiotherapist.

⚠ Rollbuddy is not suitable when the user has:

- Major problems holding balance
- Insufficient muscle strength to stand up or walk independently.
- Insufficient muscle strength to firmly grab the handle bar.

⚠ Do not stand, sit or lie on the Rollbuddy. It is not allowed to use the rollator outdoors.

⚠ Safety instructions:

LIAM Developments BV bears no responsibility for damage and/or injury due to incorrectly following the instructions.

- Maximum load on handle bar: 110 kg.
- Maximum tray/drawer load: 3 kg.
- For indoor use only on a flat, horizontal floor.

⚠ Avoid accidents:

- Only for intended use
- Use the Rollbuddy only in optimal technical condition.
- Do not alter or make technical adjustments to the construction.
- Use the Rollbuddy only on a smooth and flat surface.
- All wheels must remain in constant contact with the floor.
- When in stationary position, always apply the automatic handbrake.

Packaging:

The Rollbuddy comes folded in the box to save space and limit any risk of damage during transport.

Package contents:

- The folded frame of the Rollbuddy
- A plywood serving tray
- A drawer, with imitation leather handle
- User manual

Assembly (Fig 2-4):

The frame of the Rollbuddy **E**, **K** consists of 2 aluminum legs that are connected to each other by a central hinge **I** and a metal connecting plate **N**. When the Rollbuddy is removed from the packaging, the plate hangs between the 2 outer frame parts (Fig. 2).

Secure the metal connection plate **N** to the round connection tube **O** of the inner frame by means of the screw knob **M** (Fig. 3, 4).

Adjusting height:

Note: Always determine the height in consultation with the adviser. Slide the handle bar **G** to the desired height. Make sure that the left and right side of the height indicators **H** are on the same position. Now tighten the adjustment knobs **I** handproof.

Drawer / serving tray:

Place the drawer **C** on the metal connection plate **N**. Make sure the drawer is resting completely on the metal plate, and fits snugly under the conductors on either side of the metal plate.

The removable serving tray **B** can now be placed on the reception nooks of the frame.

Braking system (fig.5):

The Rollbuddy is equipped with an automatic braking system (standard factory setting). This means that the parking brake is always activated when the brake lever  is in the neutral position .

Fig. 5

By gradually squeezing the brake lever, braking resistance is reduced and the Rollbuddy can start moving . As soon as the brake lever is released, the parking brake is automatically reactivated.

With the brake lever in the lower position , the wheels are completely free to move.

Maintenance and cleaning:

Clean the Rollbuddy regularly. Dirt and dust can be removed with a damp cloth. If necessary, use a non-corrosive detergent against stubborn dirt. Do not use abrasives or bleach, this can cause damage or discoloration. Dry all parts with a soft, clean cloth. Clean the wheels with warm water and possibly a mild detergent. Dry them with a clean cloth.

Never use silicone spray to lubricate moving parts or to clean the rollator.

Load, measurements and technical specifications:

Max. load handle bar	110 kg
Max. carrying weight serving tray	10 kg
Max. load capacity drawer	3 kg
Total weight	5,2 kg
Folded dimensions	28x96,5x55,5 cm (LxHxB)
Dimensions folded out	71x84/97x55,5 cm (LxHxB)
Handle bar height (variable)	85 - 95 cm
Braking system	Automatisch
Wheels	ø 15 cm
Available colours	Aluminium/zwart/Koper
Article numbers/SKU	RB01ALU/RB01BLK/RB01CPR
BASIC UDI/GMN	8720726413RB01J
GTIN/UDI DI (ALUMINIUM)	8720726413008
GTIN/UDI DI (BLACK)	8720726413015
GTIN/UDI DI (COPPER)	8720726413022

Warranty:

LIAM Developments BV offers a 2-year warranty on production and assembly defects. The warranty expires when the indoor walker is not used according to the regulations specified in this document. For warranty claims, please contact LIAM Developments BV.

Rollbuddy® is a registered trademark of:

LIAM Developments BV
Jachtwagen 29
3897AM Zeewolde NL

T: 0031 (0)36 53 53 580
info@liamdevelopments.nl

© 2023 LIAM Developments BV. All rights reserved.

For more information, visit: www.rollbuddy.nl

rollbuddy[®]
www.rollbuddy.nl

LAM.
Life is about moving